LEM[®]

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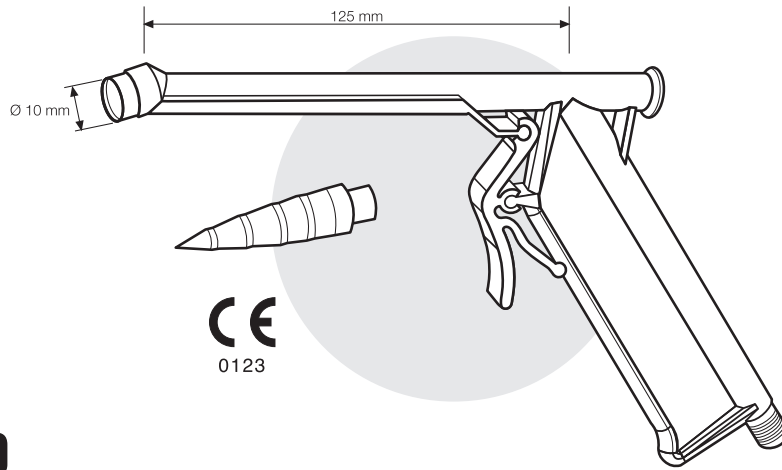
The Sapi Med LEM Disposable Haemorrhoid Banding Ligator is a unique, patented instrument. It is manufactured from plastic, designed for single-patient use, and available in two variants: one for use with suction, and the other for use with forceps. Both variants are supplied with a separable cylindrical cone (Ø 10 mm) to assist in loading the bands, an angled distal cylinder to contain the nodule, and an ergonomically-designed pistol handgrip with trigger to release the banding ring. The LEM's angled design allows the operator a clear view of the procedure.

The LEM banding ligator is designed for use with Sapi Med Self Light operative proctoscopes.

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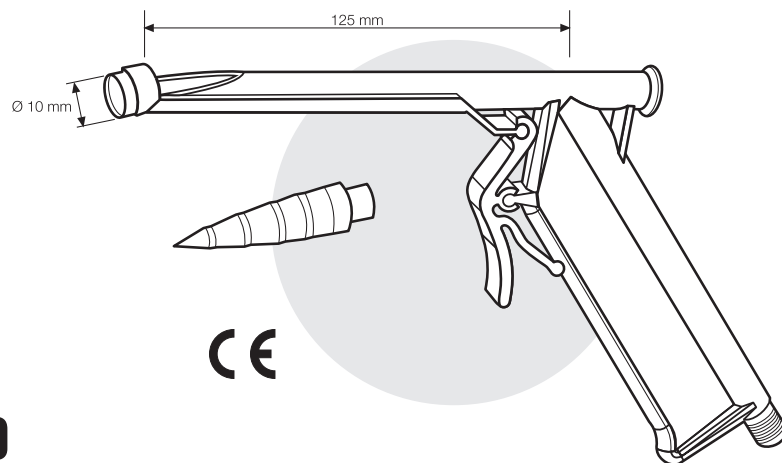
LEM for use with SUCTION

A suction tube connects the ligator device to a low-pressure suction aspirator. The haemorrhoidal nodule is drawn into the cylindrical cone-end of the device by suction, and then banded. The rate of suction is manually governed by the operator's thumb over an aspiration-regulator hole.

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LEM for use with FORCEPS

Basic design identical to suction variant. No suction aspirator is used. Instead, a cut-out opening in the shaft enables the insertion of forceps to firmly grasp the nodule.

Quantity per box: 25 pieces

Distributed by